

# AC-DS-03

## NEUTRALIZATION SYSTEM



### Space and Site Planning Overview

for using an AC-DS-03 Neutralization System with the TD 100® Automated TEE Probe Disinfector.

Please see your product manuals for specific details.

Actual Size	AC-DS-03	TD 100®
Width	12.5"	24"
Depth	16"	8"
Height	41.25"	44"
Weight	35 lbs. dry	90 lbs.
<b>Space and Site Requirements</b>		
Width	18"	36"
Depth	16"	8"
Height from floor	48"	72"
<b>Electrical Requirements</b>		
Service	120V 60 Hz	120V 60 Hz
Ratings	0.5 A	8 A
Fusing	2 A	10 A
<b>Water Supply Requirements</b>		
Delivery pressure	N/A	20-25 PSI
Flow rate	N/A	1 GPM
Temperature	N/A	50° - 80° F
Shut-Off	N/A	Required
Connection	N/A	Provided
Quality	N/A	Potable
Water usage	N/A	≈ 1 Gal
<b>Drain Requirements</b>		
Drain	Uses TD 100® drain	1-1/2" Drain Pipe & Trap Note: A special drain fitting is provided

