



TEEZyme[®]MC

DUAL ENZYMATIC CLEANER

Item No. 201910

Highly recommended for all scopes (flexible/rigid) and biopsy forceps.

Leaves instruments free of blood and protein in 2 minutes.

TEEZyme[®]MC is a dual enzymatic cleaner that removes blood, protein, mucus, vomit and fecal matter. This formulation of Protease and Amylase enzymes, with buffers and non-ionic detergents, is perfect for all scopes and instruments and extends usable lifetime. TEEZyme[®]MC will solve any problem with clogged channels, sticky forceps or clouded lenses and will leave both scopes and instruments free of any unpleasant odors or baked-on blood.

Features

- Dual enzymatic cleaner – Protease and Amylase
- Designed for universal applications: For use in all endoscope washers, washer disinfectors, ultrasonics and for manual cleaning
- Safe for use on all instruments and scopes
- Neutral pH, non-abrasive, free rinsing and biodegradable
- Will not harm any metals, plastics, rubber or corrugated tubing
- Works in all temperatures.

DIRECTIONS FOR USE

Surgical Instruments and Scopes:

1. Add one (1) ounce of TEEZyme[®]MC to one (1) gallon (8ml/liter) of water.
2. Submerge instruments and scopes to be cleaned.
For scopes: suction or flush through channels before soaking.
3. Soak for 2 minutes to remove all organic soils.
4. Rinse thoroughly with tap, distilled or sterile water.



2179 East Lyon Station Road • Creedmoor, NC 27522 USA
Phone: +1 (919) 255-9472 • Toll Free: 877-255-9472 • Fax: +1 (919) 528-3400
www.csmedicalllc.com • info@csmedicalllc.com